progetti[®]
Medical Equipment Solutions

RESCUE LIFE

*Think different, **be** different*

OPERATING ROOM

INTENSIVE CARE

EMERGENCY

TECHNOLOGY FOR DIAGNOSIS AND EMERGENCY

DESIGN

The Italian design of Rescue Life stands out for its simplicity and elegance. With **sleek lines** and an **ergonomic shape**, every detail has been meticulously refined. Rescue Life is the first professional defibrillator entirely **MADE IN ITALY**, blending tradition, style, and comfort. Its lightweight, **easy-to-carry** nature makes it the **perfect** companion **in even the most challenging situations**. Beauty, quality, and safety have been combined to define our products for years, making Rescue Life a true **standard of excellence in style and reliability**.

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TECHNOLOGY

Rescue Life results from continuous technological improvements designed to support operators in any emergency situation. This defibrillator is equipped with an internal **self-test system** that allows constant **monitoring** of the **device's functionality**, **battery status**, and **charge-discharge cycle**. It also offers the possibility of *managing data in real time* and **printing detailed reports**. The NIBP performance is ensured by **SunTech technology**.

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USABILITY

Designed to be the ultimate ally for every rescuer, Rescue Life combines advanced features with an **intuitive interface**. It boasts a robust **7-inch** TFT color LCD screen, shock-resistant and capable of up to **5 hours of continuous monitoring**. Offering a wide range of customizable configurations, the operator can select from various operating modes and additional features, including 12-lead **ECG monitoring**, **external pacemaker** (Pacer), **pulse oximeter** (SpO₂), and **non-invasive blood pressure** (NIBP). Additionally, Rescue Life offers three operating modes: Manual, AED (Automated External Defibrillator), and Advisory. In Advisory mode, the device indicates when defibrillation is recommended, allowing the operator to determine the appropriate energy level and when to charge.

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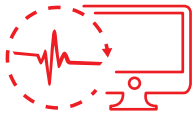
DISPLAY

The **7-inch high-resolution** (800x480 pixels) **TFT color LCD** display provides a clear and detailed view of the 7 and 12-lead ECG, simultaneously with the plethysmographic waveform. All visual messages, including notifications, warnings and alarms, are clearly shown on the display, allowing the operator to maintain **full control of the situation**. In addition, through the SETTINGS menu, it is **possible to customize and select the ECG parameters to be displayed** based on operational needs.

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FAST CONNECTION

In manual mode, the operator can choose between **reusable rigid paddles-suitable** for both adult and pediatric defibrillation or **disposable pre-gelled paddles** for external defibrillation in AED mode. Both options feature a **quick-connect system with a safety lock**, ensuring maximum reliability during use. The standard equipment includes adult/pediatric **rigid paddles**, a **5-lead ECG cable**, and **disposable electrodes** for adult patients. Internal defibrillation electrodes are available upon request.

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SENSITIVITY IN ECG DETECTION

Rescue Life automatically filters the ECG signal through two modes:

DIAGNOSTIC: ECG signal processing designed for the interpretation and diagnosis of ECG traces by qualified personnel.

MONITORING: A digitally filtered ECG signal that minimizes noise and artifacts caused by patient movement or other physiological processes.

These two modes enhance ECG detection, **assisting qualified personnel in making crucial decisions** for diagnosing heart conditions.

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OPTIONALS

Comfortable carrying bag, SpO₂ module, multifunction disposable pads for adult and pediatric, 12-lead ECG cable, pacemaker module, NIBP module, mounting bracket.

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MEMORY AND DATA MANAGEMENT

The **controls are simple, intuitive, and user-friendly**. The menu is easily accessible via the quick selector or function keys, which allow you to select the operating mode, start printing, set alarms, and adjust other parameters. The **internal memory can store up to 15 hours of ECG data**, including events, with the option to recall and **print the data at any time**. The recorder operates in both manual and automatic modes: in automatic mode, printing starts with energy loading and continues for 10 seconds after discharge.

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TECHNICAL SPECIFICATIONS

GENERAL REQUIREMENTS

Size • 369 x 240 x 340 mm
Weight • 5,5 Kg approx with battery
Electrical supply • 100 ~ 230 V _{AC} , 50/60 Hz • inverter 12V car adapter (optional) 14.4 V – 3450 mAh – 49.68 Wh Lithium-Ion battery (internal, rechargeable).
Battery Charge Time • Max. 3 hours
Battery Capacity • approximately 5.5 hours of continuous monitoring or 150 shocks at 230J with battery fully charged
Software • Voice and texts multi language
Memory • Internal memory of 30 Files, for a total of 15 hours of recording
Review and printing database • Review and printing of archived events

ENVIRONMENTAL CONDITION

Temperature range • Operational -5° ~ 46° C • Storage -20° ~ 50° C
Humidity range • 10 ~ 95%
Applied parts type • ECG type CF, Defibrillation type BF
Ingress protection • Class IP 54

DEFIBRILLATION

Waveform • Biphasic Truncated Exponential (BTE) • HiCAP technology (Large storage capacitor)
Impedance • Compensated, range +/-15% 25 ~ 200 Ohm
Charge time • < 6 sec at 230J / <5 sec at 200J (with AC)

AED MODE

Energy • Fixed energy at 200 J
Protocol • ERC 2021 CPR Guidelines with voice and text prompts
Shockable rhythms • VF with amplitude > 0.15 mV and VT with rhythm >150 bpm
Sensitivity • Shockable rhythm-VF > 95% • Shockable rhythm-VT > 75%
Specificity • Normal sinus rhythm > 99% • Asystole and other non-shockable rhythms > 95%

MANUAL MODE

Energy range • 1 - 230 J (from 1-10 J in 1 J steps; from 10 to 230 J in 10 J steps)
Commands • Multifunction trim knob. Charge and shock buttons directly on the front panel for hands free defibrillation
Paddles • Reusable adult/pediatric paddles with charge/shock command • Disposable pads and internal electrodes (optional)
Operating mode • Synchro or asynchro mode
Indicators • LEDs for battery charging and connection to AC power supply • Clear and visible backlit buttons

DISPLAY

Type • High contrast TFT color LCD 7” (800x480 pixels)
Information • Operational information, ECG traces, SpO ₂ value, NIBP values, Heart Rate values, filters, alarms and battery status
Display mode • 7-leads or 12-leads ECG traces

ECG MONITORING

Bandwidth • 0.05 to 120 Hz (-3 dB) in diagnostic mode
Speed • 5, 10, 25, 50 mm/s
Gain • 2.5, 5, 10, 20, 40 mm/mV with ECG cable. AUTO with pads
Alarm • HR max 250 bpm and HR min 0 bpm
Filters • 50/60 Hz, EMG filter, base line
Traces • 3+3+1 (I, II, III – aVR, aVF, aVL, V) with 5-lead ECG cable
Traces • 3+3+6 (I, II, III – aVR, aVF, aVL – V1 to V6) with 10-lead ECG cable
Maximum percentage amplitude of the rejected tall T-wave • ≤150%
HR • 20 a 300 bpm (± 2%)

SpO₂ (OPTIONAL)

SpO₂ range • 0 - 100%
HR range • 30 - 250 bpm
Accuracy • 70 - 100% ~ 2% for adults with finger clip sensor
Alarm • Adjustable min 50%

NIBP (OPTIONAL)

Technique • Oscillometric
Patients • Adult/paediatric/neonatal
Systolic range • Adult: 40-260mmHg, paediatric: 40-160 mmHg, neonatal: 40-130 mmHg
MAP range • Adult: 26-220 mmHg, paediatric: 26-133 mmHg, neonatal: 26-110 mmHg
Diastolic range • Adult: 20-200 mmHg, paediatric: 20-120 mmHg, neonatal: 20-100 mmHg
PR range • 30-220 bpm
PR accuracy • ±2% or ±3 bpm
Operating modes • Manual, Long Term Automatic, Stat, Service
Intervals • In Automatic mode: 1, 2, 3, 4, 5, 10, 15, 30, 60, and 90 minutes

PRINTER

Type • Integrated thermal 3 channels printer for ECG traces and events documentation including HR/SpO ₂ /NIBP values
Paper speed • 5, 10, 25, 50 mm/sec
Paper width • 58 mm
Operating modes • Manual, automatic (10” pre and post shock recording)

EXTERNAL PACEMAKER (OPTIONAL)

Type • Rectangular wave
Operating modes • Fixed, on demand
Pulse rate • 30-250 bpm, adjustable in steps of 5 bpm
Impulse duration • 22.5 ms
Pulse current • 0 to 150 mA adjustable in steps of 5mA
Amplitude • Max 150 V

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PROGETTI S.r.l.