USER MANUAL

Steel 2-wheeled walker

Product code: JMC-C 3511

Timago

1. Characteristics

The 2-wheeled walker is made of a powder-coated steel frame. It features a wide seat made of durable material. The walker is equipped with height-adjustable push handles and 6-inch rubber wheels.

The set includes: two push handles, frame with seat.

Elements of the walker:

- A. push handle
- B. locking clamp
- C. seat
- D. hinge of connecting bar

2. Application

The walker is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walker affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembly

UNFOLDING AND FOLDING THE WALKER

- **To unfold:** Pull the legs apart and press down on the connecting bar until the hinges are fully extended. Insert the push handles into the mounting openings at the desired height and secure them by locking the clamp. Unfold the seat.
- **To fold:** Lift the seat and pull the connecting bar upward until the frame is fully folded. Lower the seat back into place.

NOTE! Be careful while folding or dismantling the walker due to the risk of smashing fingers in hinges.

HEIGHT ADJUSTMENT OF HANDLES

The handle height should be adjusted so that the user can stand as upright as possible, with shoulders relaxed and elbows slightly bent. It is not advisable to place full body weight on the walker.

- To adjust the height of the handles, unlock the clamp which is located on the outside of the frame, select appropriate height and lock the clamp.

Note! The handles must be set on the same level.

4. Technical data

Overall width	51 cm
Overall depth	60 cm
Seat dimensions	40 x 22 cm
Seat height from the ground	49 cm
Handles height adjustment	82-97 cm
Weight	5,80 kg
Maximum capacity	100 kg
Frame colour	blue

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Reviews

Make visual inspection of all the joining elements (clamps, hinges) of the walker systematically, including:

- Height adjustment of handles
- Work of clamp which blocks the height of the handles
- Stability and look of the frame

Cleaning

Control the technical condition of the walker regularly:

- Clean the frame with damp cloth and dry thoroughly.

- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the walker regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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