

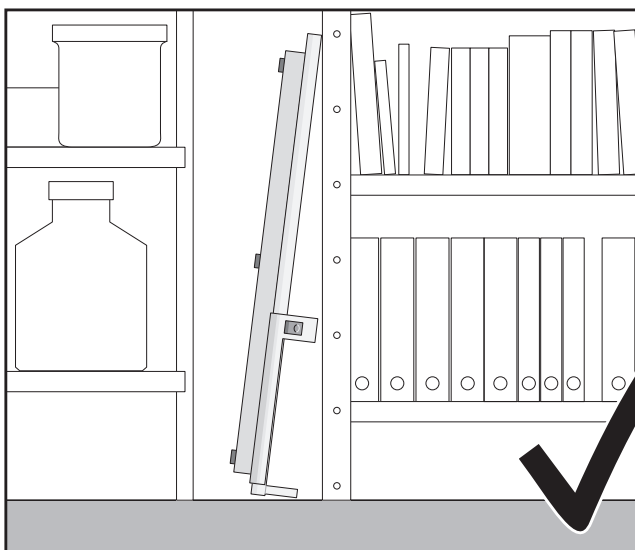
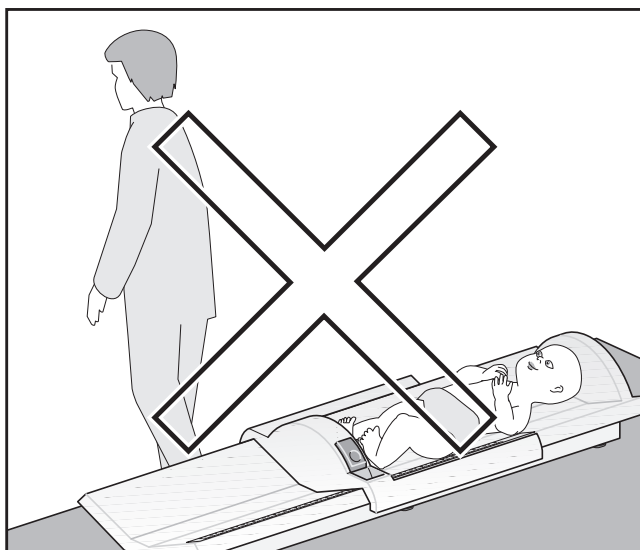
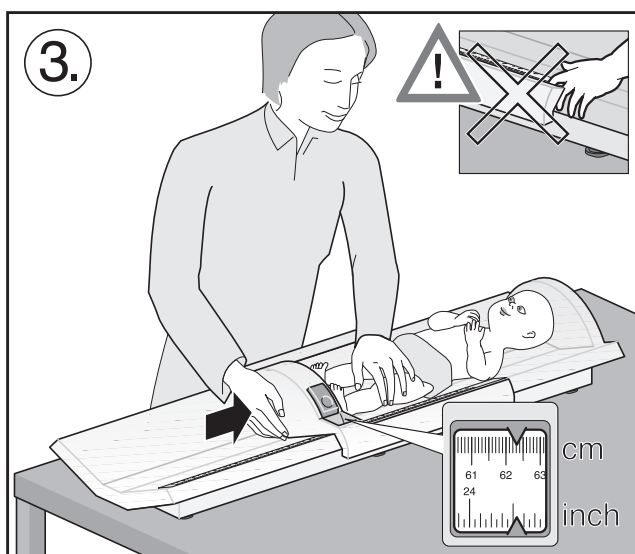
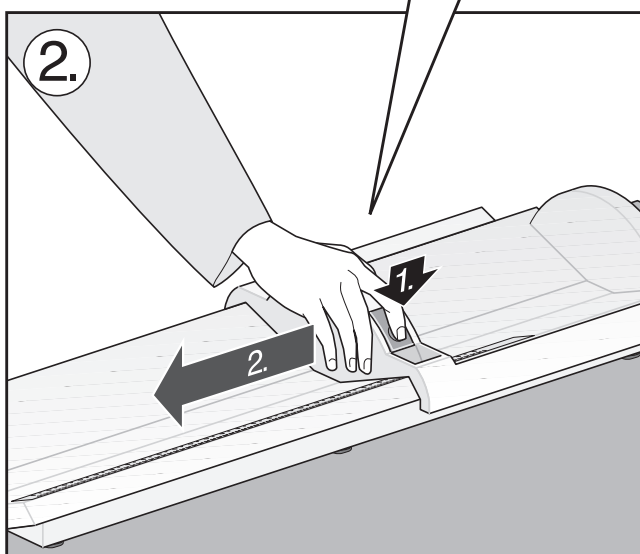
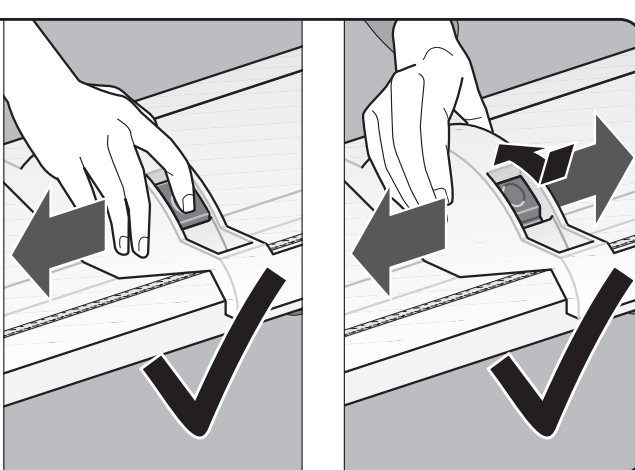
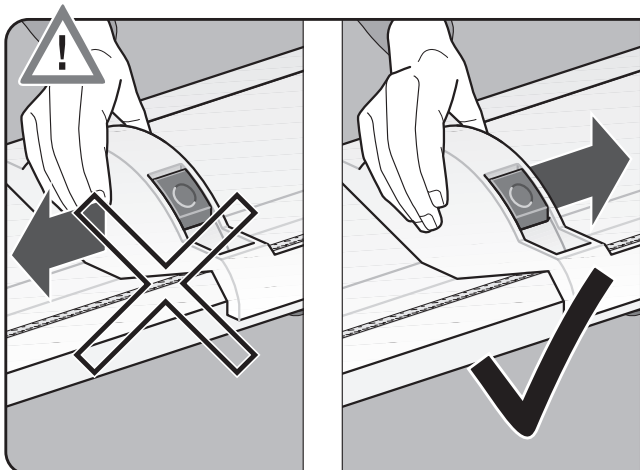


seca 416

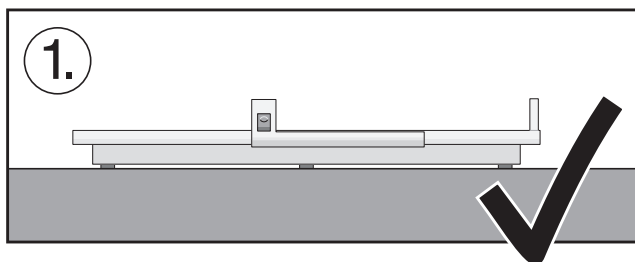

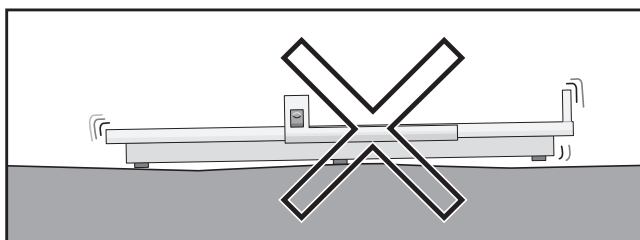

soapy solution and wipe the device over with it.

- Disinfect the device:
 - Moisten a soft cloth with disinfectant and wipe the device over with it.

Before and after each measurement	As required
• Surface of measuring mat • Head stop • Foot stop	All other parts of measuring mat

5 DISPOSAL

This device consists of high-quality raw materials which can be recycled. Dispose of the device in accordance with the local laws and regulations applicable in your country.

6 WARRANTY

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

7 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

English

1 INTENDED USE

The mechanical baby length measuring device supports physicians in decision-making regarding diagnosis or therapy, based on the patient's length or growth.

2 SAFETY PRECAUTIONS

⚠ WARNING!

Injury from falling

Measuring devices for babies are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- Make sure that the device is standing or lying on firm and even ground.
- Never leave a baby unsupervised.

3 MARKINGS ON THE DEVICE AND ON THE TYPE PLATE

Text/symbol	Meaning
UK CA xxxx	Device complies with United Kingdom directives xxxx: Notified Body for Medical Devices of the United Kingdom
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönematt Str. 2 CH-4153 REINACH
	Medical device in accordance with Regulation (EU) 2017/745

4 HYGIENE TREATMENT

⚠ WARNING!

Risk of infection




- Subject the device to a hygiene treatment at regular intervals.

⚠ NOTICE!

Damage to device

- Use alcohol-based disinfectant (e.g. 70 % ethanol) exclusively.
- Follow the instructions for use of the disinfectant.
- Do not use aggressive or abrasive cleaning agents.
- Clean the device:
 - If required, moisten a soft cloth with a mild

**Technische Daten • Technical data • Fiche technique • Specifiche tecniche •
 Datos técnicos • Tekniske data • Tekniska data • Tekniske data • Tekniset tiedot
 • Technische gegevens • Dados técnicos • Τεχνικά στοιχεία**

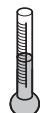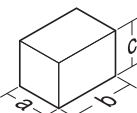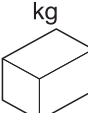				
1 mm	33 – 100 cm ± 2 mm	+10°C	a = 402 mm	3,2 kg
1/16 inch	13 – 39 inch $\pm 1/8$ inch	... +40°C	b = 1105 mm	
			c = 165 mm	

Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg
 Hammer Steindamm 3-25
 22089 Hamburg
 Germany

T +49 40 20 00 00 0
 F +49 40 20 00 00 50
 E info@seca.com

seca operates worldwide
 with headquarters in
 Germany and branches in:
 seca france
 seca united kingdom
 seca north america
 seca schweiz
 seca zhong guo
 seca nihon
 seca mexico

seca austria
 seca polska
 seca middle east
 seca suomi
 seca américa latina
 seca asia pacific
 seca danmark
 seca benelux
 seca lietuva

and with exclusive
 partners in more than
 110 countries.

All contact data at
seca.com