
**GINRAM®**

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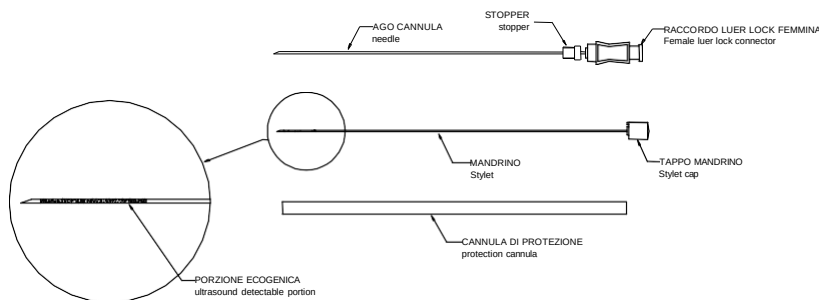GB

**Instructions for use**

Take the needle out of the pouch. Before starting the procedure, make sure that the notch of the cap is correctly inserted into its connector housing.

Place the stopper on the cannula in the right position. Remove the needle protection while holding the stopper firmly to prevent any possible coming out. Insert the needle in the abdomen on a perpendicular axis and follow ultrasound path (previously detected) until the amniotic sac is reached. Remove the mandrel and connect a proper syringe (20cc recommended) to the needle connector. Aspirate amniotic liquid by exerting a slow and moderate negative pressure on the syringe until the desired amount of amniotic fluid has been sampled. Disconnect the syringe, re-insert the mandrel into the needle and remove the needle.

Place the amniotic liquid into a dry, sterile test tube and keep it at a temperature of 37°C.

**RI.MOS. s.r.l.** via Manuzio, 15 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755  
fax +39 0535 – 25509 e-mail: [info@rimos.com](mailto:info@rimos.com) – <http://www.rimos.com>

**Caution:** Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Store at a temperature between 0°C and 40°C. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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|                                                                                                    |                                                                                                                  |                                                                                                                         |                                                                                                           |                                                                                                    |
|----------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|
| <b>LOT</b> BATCH CODE                                                                              | <b>REF</b> REFERENCE NUMBER                                                                                      |  ATTENTION. SEE INSTRUCTIONS FOR USE |  LATEX FREE            | <b>STERILE EO</b> STERILIZED BY ETHYLENE OXIDE                                                     |
|  USE BY MM/YYYY |  CONSULT INSTRUCTIONS FOR USE |  MANUFACTURER                        |  DATE OF MANUFACTURE |  DO NOT REUSE |