



Manufacturer
WATER-JEL TECHNOLOGIES LLC.
50 BROAD STREET,
CARLSTADT, NJ 07072
USA
2797

CE

Authorized Representative in the EC
SAFEGUARD TECHNOLOGIES LTD.
WILLOW GROVE
DELGANY, A63 XY89
CO WICKLOW, IRELAND

Swiss Authorised Representative
PROCANEM AG
SCHULSTRASSE 5
CH-8395 AADORF
SWITZERLAND

SAFEGUARD
MEDICAL

DTF050_WJ072_Rev 0_09-22
safeguardmedical.com

[MD] Zdravotnícký prostriedek Medicinský nástroj Medisch hulpmiddel Medical device Dispositif médical Medizinprodukt Orvosi eszköz Dispositivo medico Medisinský nástroj Wyrób medyczny Dispositivo médico Zdravotnická pomůcka Dispositivo médico Medicinteknisk produkt	Catalogue number Référence catalogue Katalognummer Katalogusszám Numero di catalogo Katalognummer Númer katalogowy Número de catálogo Katalogové číslo Número de catálogo Katalognummer	Data de validade Použito do Usar antes de la fecha Använd före	Niet opnieuw steriliseren Do not resterilise Ne pas restériliser Ne pas restériliser (uniquement pour la version stérile) Nicht wieder sterilisieren Ne sterilizálja újra Non risterrilizzare Skal ikke resterilisere Nie sterylizować ponownie Não reesterilize (apenas para a versão esterilizada) Nesterilizujte opakované No volver a esterilizar Får inte återsteriliseras
[M] Nesterilní Ikke steril Niet-steriel Non-sterile Non stérile Nicht steril Nem steril Non sterile Ikke-steril Wyrób niesterylizowany Não esterilizado Nesterilné No estéril Ikke-steril	[M] Nepoužívat opětvně Må ikke gjenbruges Niet opnieuw gebruiken Do not re-use Ne pas réutiliser Nicht wiederverwenden Ne használja újra Non riutilizzare Skal ikke gjenbrukes Nie używać ponownie Não reutilize Nepoužívať opakované No reutilizar Får inte återanvändas	Mezeleni teploty 15-25°C Temperaturgrænse 15-25°C Temperatuurlimiet 15-25°C Temperatuurlimit 15-25°C Limite de température 15-25°C Temperaturbereich 15-25°C Hőmérsékleti határérték 15-25°C Limite di temperatura: 15-25°C Temperaturgrenser: 15-25°C Zakres temperatur: 15-25°C Limitação de temperatura 15-25°C Ograniczenie temperatury 15-25°C Niet gebruiken als de verpakking is beschadigd en lees de gebruiksaanwijzing vóór het gebruik Do not use if package is damaged and consult instructions for use Ne pas utiliser si l'emballage est endommagé Nicht verwenden, wenn die Verpackung beschädigt ist und Gebrauchsanweisung beachten Tilos felhasználni, ha a csomagolás sérült, és olvassa el a használati utasítást Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso Skal ikke brukes hvis pakningen er skadet Nie używać, jeśli opakowanie jest uszkodzone i przeczytać instrukcję stosowania Não utilize se a embalagem estiver danificada Nepoužívajte, ak je obal poškodený a prečítajte si návod na použitie No utilizar si el envase está dañado, consultar las instrucciones de uso Får inte användas om förpackningen är skadad	[M] Nepoužívat jestliže je balení poškozeno Må ikke anvendes, hvis emballagen er beskadiget – læs vejligst brugsanvisningen Niet gebruiken als de verpakking is beschadigd en lees de gebruiksaanwijzing vóór het gebruik Do not use if package is damaged and consult instructions for use Ne pas utiliser si l'emballage est endommagé Nicht verwenden, wenn die Verpackung beschädigt ist und Gebrauchsanweisung beachten Tilos felhasználni, ha a csomagolás sérült, és olvassa el a használati utasítást Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso Skal ikke brukes hvis pakningen er skadet Nie używać, jeśli opakowanie jest uszkodzone i przeczytać instrukcję stosowania Não utilize se a embalagem estiver danificada Nepoužívajte, ak je obal poškodený a prečítajte si návod na použitie No utilizar si el envase está dañado, consultar las instrucciones de uso Får inte användas om förpackningen är skadad
[LOT] Kód dávky Batchcode Batchcode Batch code Numéro de lot Chargennummer Tételkód Codice lotto Partinummer Kod partii Número de lote Kód šarže Código de lote Batchnummer	[LOT] Kód dávky Batchcode Batchcode Batch code Numéro de lot Chargennummer Tételkód Codice lotto Partinummer Kod partii Número de lote Kód šarže Código de lote Batchnummer	[LOT] Sterilizováno ozářením Steriliseret ved hjælp af bestråling Gesteriliseerd met behulp van bestraling Sterilised using irradiation Stérilisé par irradiation (uniquement pour la version stérile) Durch Bestrahlung sterilisiert Besugárzással sterilizálva Sterilizata mediante irradiazione Steriliserat med strålning Wysterylizowano przez napromienienie Esterilizado com irradiação (apenas para a versão esterilizada) Sterilizované ožiarením Esterilizado mediante radiación Steriliserad med bestrålning	[LOT] Sterilizováno ozářením Steriliseret ved hjælp af bestråling Gesteriliseerd met behulp van bestraling Sterilised using irradiation Stérilisé par irradiation (uniquement pour la version stérile) Durch Bestrahlung sterilisiert Besugárzással sterilizálva Sterilizata mediante irradiazione Steriliserat med strålning Wysterylizowano przez napromienienie Esterilizado com irradiação (apenas para a versão esterilizada) Sterilizované ožiarením Esterilizado mediante radiación Steriliserad med bestrålning
[REF] Katalogové číslo Katalognummer Catalogusnummer	[REF] Katalogové číslo Katalognummer Catalogusnummer	[REF] Katalogové číslo Katalognummer Catalogusnummer	[REF] Katalogové číslo Katalognummer Catalogusnummer

EN

BURN GEL

PRODUCT NAME: Burn Gel

PRODUCT CLAIMS: Immediate first aid for minor burns and scalds. Fast burn relief. Cools and soothes. Helps prevent infection.

INTENDED USE: For use as first aid treatment on a burn by medical professional and laypersons alike.

INSTRUCTION FOR USE: Apply a generous layer to affected area. Do not rub into skin. The gel is most effective when applied in a thick layer and allowed to remain in contact with skin for a minimum of 30 minutes. Re-apply as often as necessary. Cover if necessary.

RESIDUAL RISKS / WARNINGS: Remove and seek medical advice if rash develops. Seek emergency medical care for severe burns. Device is for external use only. Do not apply compression around gel. Do not use after expiry provided on packaging. Dispose of as clinical waste as per local protocols.

USER PROFILE: Intended to be used by medical professionals and laypersons alike.

PATIENT PROFILE: For patients with minor burn and scald injuries.

PERFORMANCE CHARACTERISTICS: N/A

CONTRAINDICATIONS: N/A

CONTENTS: Depends on the size of product.

REPORTING: Any serious incident that has occurred in relation to this device should be reported to manufacturer (vigilance@safeguardmedical.com) and applicable national competent authority.

BURN DRESSING

PRODUCT NAME: Burn Dressing

PRODUCT CLAIMS: Emergency first aid for

burns. Stops the burning process. Cools the burn. Helps prevent contamination and reduces wound adhesion.

INTENDED USE: Intended for use as emergency first aid treatment on a burn.

INSTRUCTION FOR USE: Tear open packaging. Remove Burn Dressing and apply immediately to burn area. Wrap area without applying pressure. Leave dressing in place. Seek further medical assistance.

RESIDUAL RISKS / WARNINGS: Remove and seek medical advice if rash develops. Seek emergency medical care for severe burns. Do not re-use due to risk of contamination. Sterility is guaranteed unless the pouch is damaged or opened prior to use. Do not use if sterile barrier is compromised. Do not re-sterilise (for sterile version). Do not use if packaging damaged (non-sterile version). Device is for external use only. Do not apply compression around dressing. Do not use after expiry provided on packaging. Dispose of as clinical waste as per local protocols and risk of contamination if device is re-used.

USER PROFILE: Intended to be used by medical professionals and laypersons alike.

PATIENT PROFILE: For patients with burn and/or scald injuries.

PERFORMANCE CHARACTERISTICS: N/A

CONTRAINDICATIONS: N/A

CONTENTS: 1 dressing.

REPORTING: Any serious incident that has occurred in relation to this device should be reported to manufacturer (vigilance@safeguardmedical.com) and applicable national competent authority.

FIRE BLANKET

PRODUCT NAME: Fire Blanket

PRODUCT CLAIMS: Stops the burning process. Removes heat from burn site. Extinguishes small fires.

INTENDED USE: Intended for use as emergency first aid treatment on a burn.

INSTRUCTION FOR USE: Burn Wrap and Fire Blanket in cannister: To remove lid: Firmly grasp handle on lid and quarter-turn counter-clockwise. Lift lid to reveal blanket. Tear open inside bag to remove blanket. Lay burn victim down, cover burn area with blanket and gently conform. Leave in place and seek further medical attention. Burn Wrap and Fire Blanket in pouch: Tear open inside bag and remove blanket. Lay burn victim down, cover burn injured area with blanket and gently conform. Leave in place and seek further medical attention.

RESIDUAL RISKS / WARNINGS: Remove and seek medical advice if adverse reaction occurs. Seek emergency medical care for severe burns. Do not re-use due to risk of contamination. Do not use if packaging damaged. Device is for external use only. Do not apply compression around blanket. Do not use after expiry provided on packaging. Dispose of as clinical waste as per local protocols.

USER PROFILE: Intended to be used by medical professionals and laypersons alike.

PATIENT PROFILE: For patients with burn and scald injuries.

PERFORMANCE CHARACTERISTICS: N/A

CONTRAINDICATIONS: N/A