



PROFESSIONAL MEDICAL PRODUCTS

TAVOLA SPINALE VALU

VALUE SPINAL BOARD

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

I. Technical Features:

1. Product size (L x W x H): 183 x 43 x 7 cm
2. Packing size: 186 x 47 x 7 cm
3. N.W.: 8 kg - G.W.: 9 kg
4. Load bearing: less than 159 kg

II. How to use:

The spine board used for transporting patients with fractures. High-density PE plate for light weight, strong and economic. This allows for the patient to be quickly and gently placed onto the stretcher, operation table or bed. The stretcher is then withdrawn from beneath the patient.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

IV. Cautions:

1. When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
2. In general operation, be careful not to out torn the stretcher surface.

V. Maintenance:

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

VI. Storage & transportation:

1. Store this product in the place of damp proof & non-corrosion.
2. Common transporting vehicle can carry this product.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		Unique device identifier
	Imported by				

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Gima 34092



Zhangjiagang Tengda Machinery Manufacturer Co., Ltd
No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made in China



TD010162A



SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA
Capelle aan den IJssel, The Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

